SKU# 200-037

Sales Inquiries: John Sperrazzo
(858) 361-4495 john@zshieldsusa.com

Reusable Protective Face Shield 2.0

Personal protective equipment (PPE) designed to help protect your face, mouth and eyes from spray and splatter.

Product Features:

- Fast assembly and easy cleaning
- Elastic band offers secure fit
- Scratch resistant
- Anti-fog

MADE IN THE USA

Each set includes one each of these component parts:

Headset

Dimensions

X - 8.16" Y - 7.78" Z - 0.47"

Materials

Profax 6323 (PP Homopolymer)
Profax 6523 (PP Homopolymer)
Profax PD702 (PP Homopolymer)

Usage Multi-Use
Sterilization NON-STERILE

Sterilization/Cleaning Instructions

The autoclave must reach and maintain a temperature of 121° C for at least 30 minutes by using saturated steam under at least 15 psi of pressure (note: applies to headsets only).

Flex Fit

The headset is designed for an adjustable fit. If you gently bend the arms outward, you'll be able to get a more comfortable fit for larger head sizes.

Visor

Dimensions

8.75" L

Materials

Polycarbonate
PETG
APET

Usage Multi-Use
Sterilization NON-STERILE

Sterilization/Cleaning Instructions

- Wipe down with soap and warm water
- UV Sterilization

Please Recycle


PET
VISOR

PP
HEADSET

Application Band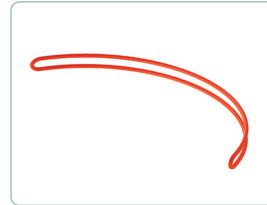

Dimensions

Standard Elastic Band

Materials

Organic Rubber


Usage Single-Use
Sterilization NON-STERILE

Safety Tips


- Each headset should be cleaned between uses
- Parts should not be shared between persons without submitting for sterilization
- Always practice good hygiene

Assembly Instructions


How to Assemble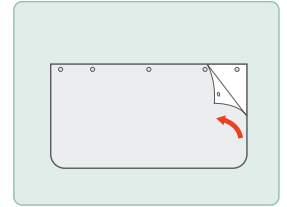

1. Remove items from box and place on a clean surface. Peel off protective cover from visor.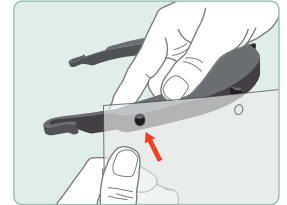


2. Ensure headset is positioned correctly with "top side" facing up. Align holes in visor with pins in headset (left to right). Attach visor to headset by snap-fitting each prong to hole.

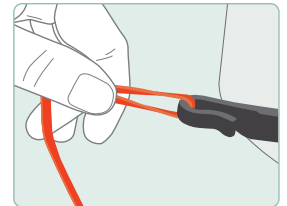3. Loop application band through hooks in headset.

4. Apply protective face shield to head and secure in place with application band.

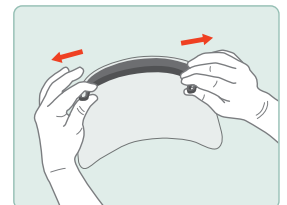5. For larger head sizes, gently bend the arms of the headset outward for a more comfortable fit.

Personal Protective Equipment EUA

On the basis of this determination, the Secretary of Health & Human Services declared that circumstances exist justifying the authorization of emergency use of personal respiratory protective devices during the COVID-19 outbreak pursuant to section 564 of the FD&C Act, subject to the terms of any authorization issued under that section.

If you have any questions, please email: CDRH-NonDiagnosticEUA-Templates@fda.hhs.gov

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